Nord BONDING SE Universal

EN	Light curing self etching universal adhesive
DE	Lichthärtendes selbstätzendes Universaladhäsiv
BG	Лекo втвърдяващо се ецващо универсално лепило
FR	Adhésif universel auto-mordançant photopolymérisable
EL	Ελαφριά σκληρυνση αυτοκόλλητη καθολική κόλλα
IT	Adesivo universale fotopolimerizzabile automordanzante
PL	Światłoutwardzalny samotrąwiający uniwersalny klej
ES	Adhesivo universal fotopolimerizable autograbante
TR	İşıkla sertleşen, kendinden aşındırıcı universal yapıştırıcı

INSTRUCTION FOR USE

DESCRIPTION

Nord BONDING SE Universal is a light curing, bioinert, single component universal self-etching adhesive which incorporates aspects of etching and bonding in a single bottle.

COMPOSITION

Methacrylate mixture 40-60%, MDP monomer 15-25%, alcohol 10-25%, coinitiator ≤7%, water ≤5%, silicon dioxide ≤3%, photoinitiator ≤2%, inhibitors ≤1%.

Nord BONDING SE Universal does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

light curing	10 s
shear bond strength (enamel)	19.6±1.0 MPa
shear bond strength (dentin)	23.2±1.0 MPa

INTENDED PURPOSE AND CLINICAL BENEFITS

Nord BONDING SE Universal helps to restore/improve aesthetic appearance of restorable tooth; helps to restore/maintain dental function of restorable tooth; helps to protect biological structures of restorable tooth and adjacent tissues.

CLINICAL INDICATIONS

- To bond light or dual curing composites and compomers to tooth structure (dentin and enamel);
- To bond inlays, onlays, crowns, bridges, veneers in combination with a light or dual curing luting materials;
- To improve retention of a restoration.

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

Nord BONDING SE Universal should not be used with products containing eugenol as eugenol may disturb polymerization process.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, Nord BONDING SE Universal may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract).

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth.

INTENDED USER

Nord BONDING SE Universal is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental adhesive. There is no need for specific training.

STERILITY

Nord BONDING SE Universal is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

Nord BONDING SE Universal is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of adhesive is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

CONSUMABLE COMPONENTS AND ACCESSORIES

No accessories are supplied with the device. Consumables, such as micro applicators, cup for application, are supplied with the device.

INSTRUCTION FOR USE

1. Prepare cavity as always. Cleaning with oil-free prophylaxis paste, such as Nord FASTE is recommended. For deep cavities use calcium hydroxide liner or glass ionomer base lining cement.
2. The surface must be not contaminated prior to applying the adhesive.

3. Shake the bottle well before each use.
4. Delicate push on bottle and dispense few drops of adhesive into dispensing cup. Do not force bottle.
5. Using a fully saturated micro applicator tip, apply Nord BONDING SE Universal on the prepared surface.
NOTE! Turn off or turn away the lamp from the working surface so the material does not harden.
6. Under light pressure rub the applied adhesive using the same micro applicator for 20 seconds.
7. Blow the surface with strong air force for 5 seconds to evaporate excess solvent. Use a vacuum aspirator to prevent the bonding liquid from scattering.
8. Curing with light for 10 seconds. Use LED polymerization lamp with light intensity 1200mW/cm². Some lamps with higher intensity could require less time of polymerization, follow manufacturer's instruction for use.
9. Apply luting cement (indirect restorations) or light curing restorative material (direct restorations), such as Nord NANO, Nord FLOW and finish restoration.

WARNINGS

After the desired amount of adhesive extruded, immediately close the bottle cap. Nord BONDING SE Universal is a highly flammable liquid. When removing Nord BONDING SE Universal from refrigerator after storage, leave the bottle at room temperature for some time (about 15 minutes) until warm-up. For non-cut enamel surfaces, pre-etching with phosphoric acid gel, such as Nord GEL is always recommended. Do not use Nord BONDING SE Universal on exposed pulp. Do not use Nord BONDING SE Universal for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. Nord BONDING SE Universal does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF ON SKIN OR MUCOSA: Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. IF SWALLOWED: Rinse mouth. Call a Poison Center or doctor/physician if you feel unwell. IF INHALED: Remove person to fresh air and keep comfortable for breathing.

Wash hands thoroughly after handling. Use only in a well-ventilated area. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

Precautions to be taken in the event of changes in the performance of the device:

If noticed any changes in the performance of the device, immediately remove applied material with suitable dental instrument. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center.

SHELF-LIFE

Shelf-life of Nord BONDING SE Universal is 2 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

STORAGE

Keep product tightly closed in dry well-ventilated place at 2-8°C (preferable in a refrigerator). Protect from direct sunlight. Keep away from heat/sparks/open flames/hot surfaces. — No smoking. Do not freeze. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

Nord BONDING SE Universal is safe and performs as intended if it is used in accordance to manufacturer's instruction for use. Summary of safety and clinical performance will be introduced in EUDAMED as soon as it will start work.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

PACKAGING

REF NRDBSTP 5ml bottle, 5pcs micro applicators, cup

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