Nord CORE

EN	Dual curing core build-up and post cementation material
DE	Dualhärtendes Stumpfaufbau- und Stifbefestigungsmaterial
BG	Материал за двойно втвърдяване на сърцевината и след циментиране
FR	Matériau de construction de noyau et de post-cémentation à double durcissement
EL	Υλικό σκασόρευσης διπλού πυρήνα και μετά από τσιμέντο
IT	Materiale per la ricostruzione del moncone e post cementazione a polimerizzazione duale
PL	Podwójnie utwardzalny materiał do odbudowy zrębu i po zacementowaniu
ES	Material de curado dual para reconstrucción de muñones y post cementación
TR	Çift kürlenen kor oluşumu ve simantasyon sonrası malzeme

INSTRUCTION FOR USE

DESCRIPTION

Nord CORE is a dual curing, bioinert, radiopaque highly aesthetic material which ensures strong adhesion to tooth surface and long lasting restoration. Nord CORE is mixed automatically.

COMPOSITION

Base: dental glass grinded based on silica and barium 50-70%, methacrylate mixture 30-50%, silicon dioxide 1-5%, initiator <1%, opacifier <1%, inhibitor <1%. Catalyst: dental glass grinded based on silica and barium 50-70%, methacrylate mixture 30-50%, silicon dioxide 1-5%, coinitiator <1%, stabilizer <1%, photoinitiator <1%, opacifier <1%, inhibitor <1%, pigment <1%.

Nord CORE does not contain medicinal substance, including human blood or plasma derivate; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

dual curing	light and chemical
flexural strenght	92.3±4.8 MPa
water sorption	18.49±0.15 µg/mm ³
water solubility	0.75±0.26 µg/mm ³

INTENDED PURPOSE AND CLINICAL BENEFITS

Nord CORE restores/improves aesthetic appearance of restorable tooth; restores/maintains dental function of restorable tooth; protects biological structures of restorable tooth and adjacent tissues.

CLINICAL INDICATIONS

For core build-up and post cementation.

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

Nord CORE should not be used with products containing eugenol as eugenol may disturb polymerization process.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, Nord CORE may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract irritation).

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth.

INTENDED USER

Nord CORE is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental materials. There is no need for specific training.

STERILITY

Nord CORE is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

Nord CORE is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of material is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

CONSUMABLE COMPONENTS AND ACCESSORIES

No accessories are supplied with the device. Consumables, such as automix tips, intraoral-root canal tips, endo tips, are supplied with the device.

INSTRUCTION FOR USE

PREPARATION:

1. Prepare contact surface using standard technique. The surface must be thoroughly rinsed and dried. In areas close to pulp use calcium hydroxide liner or glass ionomer base lining cement.
2. Place a transparent matrix and fix it in place.
3. Apply layer of etch, such as Nord GEL on all surface to be etched. Leave etch in place for 15 seconds (dentine), 30 seconds (enamel). Rinse with water and dry with air. Avoid over drying dentin. After drying, the surface must be chalky and do not contaminated prior to applying the adhesive. If the surface becomes contaminated with saliva, rinse and dry again and re-etch if necessary, follow manufacturer's instruction for use.
4. Apply a thin layer of adhesive, such as Nord BONDING LC immediately onto etched surface, follow manufacturer's instruction for use.
5. The working area must be kept dry.

6. Remove Nord CORE syringe cap.
7. Promptly and carefully attach the automix tip to the syringe.
8. Test flow of material from automix tip before using.
9. Squeeze out a 1-2mm of material onto a mixing pad to equal the base paste and catalyst paste in the automix tip. Do not use the paste which has been squeezed out.

POST CEMENTATION AND CORE BUILD-UP:

1. Using automix syringe and appropriate tip, apply Nord CORE directly into cavity. Keep the tip immersed in the material while dispensing to avoid air bubbles.
2. Insert prepared post. Light cure for 10 seconds to fix the position of the post.
3. Continue to apply the material around the post and form a core. Working time is about 165 seconds at 23°C.
4. For easier manipulation of a large amount of material, it is recommended to build-up core incrementally and light cure each layer for 5 seconds.
5. As final setting light cure for 20-40 seconds from labial (buccal) and lingual sides. Use LED polymerization lamp with light intensity 1200mW/cm². Some lamps with higher intensity could require less time of polymerization, follow manufacturer's instruction for use.
6. Chemical curing is about 3-4 minutes at 37°C.

FINISH:

1. Leave mixing tip on the syringe and change the tip prior to the next use.
2. When material has hardened fully, remove a matrix if was used.
3. Finish using standard technique.

WARNINGS

Increased humidity and/or temperature will reduce the working/setting time. Vice versa decreased humidity and/or temperature will prolong the working/setting time. The device is sensitive to light. The working time significantly reduces during application under intensive lighting. When removing Nord CORE from refrigerator after storage, leave the material at ambient temperature for some time until warm-up. Do not use Nord CORE for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. Nord CORE does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS!

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF ON SKIN OR MUCOSA: Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. IF SWALLOWED: Rinse mouth. Call a Poison Center or doctor/physician if you feel unwell. IF INHALED: Remove person to fresh air and keep comfortable for breathing. Wash hands thoroughly after handling. Use only in a well-ventilated area. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

Precautions to be taken in the event of changes in the performance of the device:

If noticed any changes in the performance of the device, immediately remove applied material with suitable dental instrument. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center.

SHELF-LIFE

Shelf-life of Nord CORE is 2 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

STORAGE

Keep product tightly closed in dry well-ventilated place at 2-8°C (preferable in refrigerator). Protect from direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

Nord CORE is safe and performs as intended if it is used in accordance to manufacturer's instruction for use. Summary of safety and clinical performance will be introduced in EUDAMED as soon as it will start work.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

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